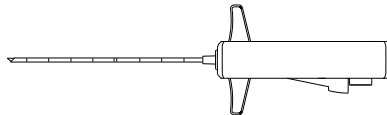
MEDONE ULTRA

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INTENDED USE

ENGLISH

MEDONE ULTRA biopsy system has been designed to be used for histological biopsy of soft tissues (such as breast, kidney, liver, prostate and various soft tissue masses). It is not intended for use in bone biopsy.

Warning: Test the device by firing into the air may damage the sharpening of the cannula and/or the mandrel tip. Do not keep fingers or other objects in front of the device as the needle will advance 2.5cm rapidly, during activation.

INSTRUCTIONS FOR USE

Preliminary activity:

- Using appropriate aseptic technique, remove the device from its package and prepare the biopsy site.
- Disinfect the skin and proceed with administering an adequate level of anaesthetic.
- If necessary, use a scalpel to make a localized incision into the skin, in order to facilitate access of the needle cannula.

Step 1: Preparation of Medone Ultra biopsy system

Load the Medone Ultra biopsy system by pulling back the loading levers to the end of the stroke and release twice; the first time to load the cutting cannula, the second time to load the mandrel.

Step 2: Positioning of the Needle

Position the needle tip at or near the lesion to be biopsied. It is recommended to use image guidance systems to reach the desired area.

Step 3: Fire the Biopsy System

Two firing sequence methods are available once the biopsy system is loaded:

1. Delayed Mode (semiautomatic method) - By pressing the D button, the mandrel will trip and advance in the sampling area. The operator uses this delay firing method to verify the position of the biopsy notch in the target area prior to pressing the A button. Pressing button A activates the cannula that advances, cutting and capturing the specimen in the biopsy notch.
2. Automatic mode - The operator can choose to use the fully automatic method, by simply pressing button A first. In this way mandrel and cannula will advance automatically in sequence for the collection and capture of the specimen.

A = Automatic Mode

D = Delayed Mode (Mandrel only)

Step 4: Biopsy Specimen

To retrieve the collected tissue sample, hold the device with the A and D firing buttons facing up and pull back and release completely the loading levers once. This action will retract the cannula exposing the mandrel biopsy notch containing the specimen.

Step 5: Multiple sampling

After the tissue sample is removed from the biopsy notch, additional tissue samples from the same patient may be obtained.

Pull back the loading levers a second time to retract the mandrel within the cannula and to make the device ready for use again. Repeat Steps 3 to 5.

In the automatic mode, the Medone Ultra biopsy system is designed to fire mandrel and cannula in rapid succession. Please exercise caution before proceeding as the device will advance 2.5 cm at the time of activation.

Biopsy procedure with coaxial introducer needle:

1. Remove protective coaxial introducer needle cap.

2. Slowly insert the coaxial introducer needle into the tissue up to reach the proximity of the biopsy area or the proximity of tissues at risk of injuries.
3. Remove the mandrel by turning the handle counter-clockwise.

4. NOTE: Model provided of additional blunt tip stylet

In case the zone under examination is found to be close to a critical area (arteries, veins, internal organs, etc.) it is possible to use a special atraumatic blunt tip stylet in order to reduce the risks of involuntary damaging of such areas.

- After having removed the trocar tip mandrel, insert the additional blunt tip stylet into the coaxial cannula and turn it clockwise to engage with the coaxial needle cannula hub. Proceed with the penetration to the margins of the lesion that will be submitted to biopsy evaluation.
 - Remove the blunt tip stylet by turning the handle counter-clockwise.
5. Insert the previously charged Medone Ultra biopsy system inside the coaxial cannula and proceed to the sample collection (refer to the biopsy system instruction for use described in the previous section).
 6. Remove the Medone Ultra biopsy system and insert it again to perform multiple biopsies.
 7. At the end of procedure, remove all devices and accessories paying attention to the necessary precautions for potentially contaminated materials.
 8. Throw away after the use and discharge together with the hospital waste following the laws in force.

LIMITATIONS AND WARNINGS

- Usage allowed only to qualified physicians. MEDAX declines every responsibility for damages caused to the patient when the device is used in an improper way, with wrong maneuvers or if handled by non qualified personnel.
- The procedure described is for guidance only: the use of the device must be based on the clinical training of each physician.
- Before using, inspect the needle to exclude the presence of damaged tip, bending or other imperfections that would prevent proper function. If not, do not use the device.
- Unusual resistance against the mandrel during the biopsy maneuver may cause the mandrel to bend at the specimen notch and thus interfere with the needle function.
- The device is sterilized by ethylene oxide and not to be resterilized or reconditioned in any way.
- Single-use/single-patient device: re-use of the device may cause cross infection to patient and/or user.
- Verify the validity of the product before use.
- The product must not be used if the package is unsealed, damaged or wet.
- Keep in a fresh and dry place, avoid the exposure to light and high temperature.
- Do not use in magnetic resonance (MR) environment.

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